

One Step Strip Style HCG Urine Pregnancy Test

THE TEST

One Step Strip Style HCG Urine Pregnancy Test is a test for the determination of HCG (Human Chorionic Gonadotropin) concentration in urine specimens and in turn for the early detection of pregnancy. This test strip is used to obtain a visual result.

TEST PROCEDURE


1. When you are ready to begin testing. Open the sealed pouch by tearing along the notch. Remove the test from the pouch.
2. Immerse the strip into the urine with the arrow end pointing towards the urine. Do not immerse pass the MAX (maximum) line. Take the strip out after 3 seconds and lay the strip flat on a clean, dry, non-absorbent surface (e.g. mouth of the urine container).
3. Wait for colored bands to appear. Depending on the concentration of hCG in the test specimen. Positive results may be observed in as short as 40 seconds. However, to confirm negative results, the complete reaction time (5 minutes) is required. Do not read results after 10 minutes.

READING OF TEST RESULT

Negative: Only one colored band appears on the control region. No apparent band on the test region. There is no pregnancy.

Positive: Distinct colored bands appear on the control and test regions. Both test line and control line indicate that you are pregnancy. Color intensity of the test bands may vary. Different stage of the pregnancy have different concentration of hCG level.

Invalid: No visible band at all. Repeat test with a new test strip.

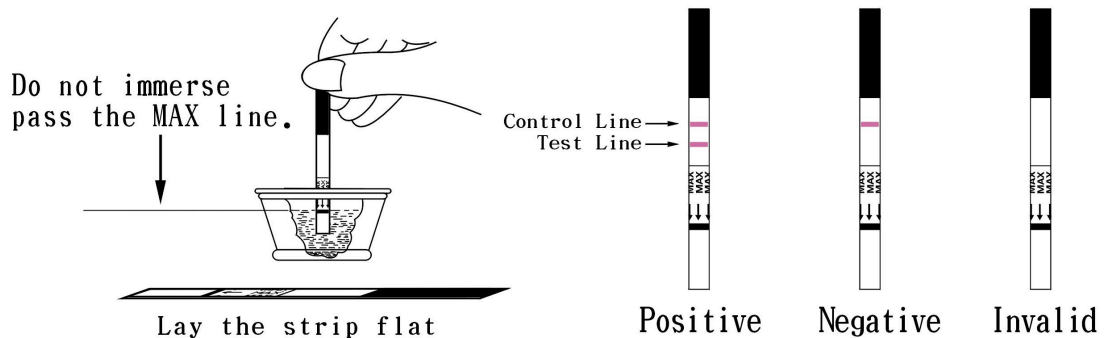

STORAGE AND STABILITY

The test strip can be stored at room temperature (4 to 30°C) in the sealed pouch for duration of shelf life. The test strip should be kept away from direct sunlight, moisture and heat.

STORAGE AND STABILITY

- * For in vitro diagnostic use only.
- * Do not use test strip beyond expiry date.
- * The test strip should not be reused.



EU Representative:
SHANGHAI INTERNATIONAL HOLDING CORP. GMBH
Eiffestr. 80
20537 Hamburg, Germany